

**American Heart Association
BLS Training Center
WE R CPR LLC
Policy and Procedure**

Index Title BLS Manikin Decontamination/Cleaning

Purpose: To ensure that all BLS manikins are cleaned in accordance with American Heart Association guidelines.

Scope: TC Coordinator, Lead Instructor, TC Faculty and BLS Instructors

Policy: All BLS manikin decontamination/cleaning will be done using current American Heart Association guidelines.

Procedure:

1. Remove all non-disposable parts and vigorously wash with soap and water using a scrub brush.
2. Thoroughly rinse all non-disposable parts and submerge them in a solution of ¼ cup bleach to 1 gallon of water. Soak for 10 minutes.
3. Thoroughly rinse all non-disposable parts and place on clean towels to dry.
4. Once a month wash all of the manikin's clothes.
5. Wipe manikin face and mouth with alcohol between each use: Leave the surface wet for at least 30 seconds before wiping

Signature/date

Signature/date